
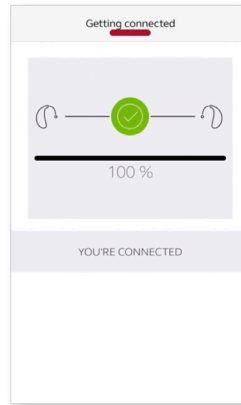
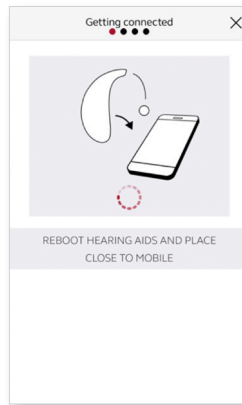


## How to pair your hearing aids with your Android™ device

Absolute Hearing Solutions  
614-452-4280

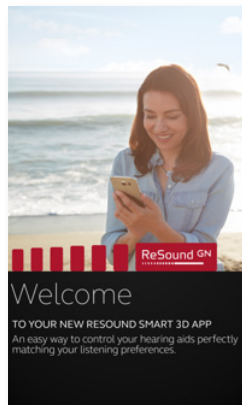

1. Download the ReSound app, open it and tap **Get Started**. Select **Agree** to the Terms and Conditions.
2. If prompted, continue by enabling location services by tapping **OK**, and ensure Bluetooth is on.

3. The app will guide you through the process. Reboot your hearing aids as required by turning them off and on again.
4. Pairing is complete.

NOTE: If you have previously paired hearing aids, you need to unpair them in your phone's Bluetooth menu before you pair them again.

### Manufacturer according to FDA:

**GN ReSound North America**  
8001 E Bloomington Freeway  
Bloomington, MN 55420  
USA  
1-888-735-4327  
resound.com

### ReSound Government Services

8001 E Bloomington Freeway  
Bloomington, MN 55420  
USA  
1-800-392-9932  
resound.com/veterans

### Manufacturer according to Health Canada:

**ReSound Canada**  
303 Supertest Road  
Toronto, ON M3J 2M4  
Canada  
1-888-737-6863  
resound.com

© 2019 GN Hearing A/S. All rights reserved. ReSound is a trademark of GN Hearing A/S. Apple, the Apple logo, iPhone, iPad, iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc.



401279011 Rev A 2019.11



## Get connected

The ReSound Smart 3D™ and Smart™ apps

GN Making Life Sound Better  
FOR 150 YEARS



## Introduction


Thank you for choosing our products. We have created this app to make it easy for you to use and control your hearing aids from your smartphone or mobile device.

For information on which smartphones and mobile devices are compatible with the app, visit: [resound.com/compatibility](https://resound.com/compatibility).

Before you can start using the app, you must pair your smartphone or mobile device with your hearing aids. This is easily done – just follow the steps below.

## How to pair your hearing aids with your iOS device (iOS 13 or later)

1. Turn your hearing aids off.
2. Confirm **Bluetooth** is on by going to the **Settings** menu on your iOS device. The slider turns green when you activate Bluetooth.

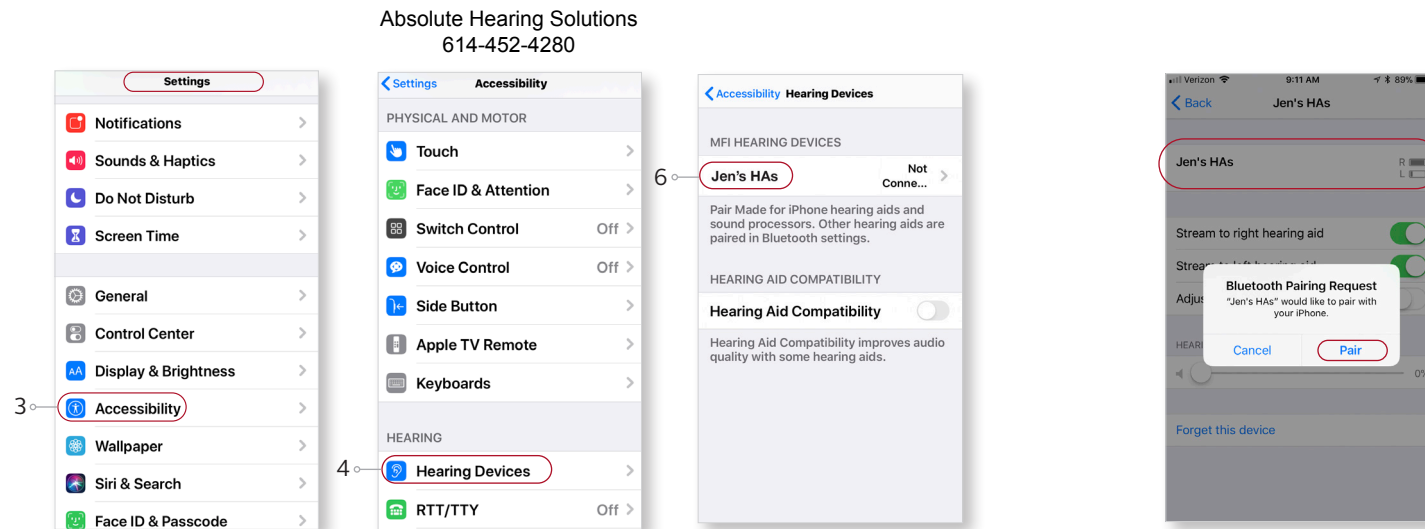

3. Go back to the main **Settings** menu and scroll down to tap on **Accessibility**.

4. Scroll down to **Hearing Devices**.
5. Turn your hearing aids on. Now, your iOS device will be able to detect your hearing aids.

6. Once connected, select your hearing aids under Hearing Devices.
7. Tap on **Pair**.
8. Go to the **App Store**, and in the **Search** field on top enter the name of the app.
9. Download the app, open it and tap **Get Started**. Follow the instructions on the screen to connect to the app.
10. You are now ready to go.

## If you need to re-pair your iOS device

1. Start by removing the old pairing by going to the **Settings** menu.
2. Tap **Accessibility**.
3. Select **Hearing Devices** and tap **Forget this device**.
4. Turn your iOS device off and on again and re-pair the hearing aids to the device. If you need more information, contact your hearing care professional.

