



Absolute Hearing Solutions

Keep Your Hearing Aids Performing Their Best



How to Replace Starkey Hear Clear™ Filters




1

Take one replacement stick.



2

Use removal end to remove old filter.

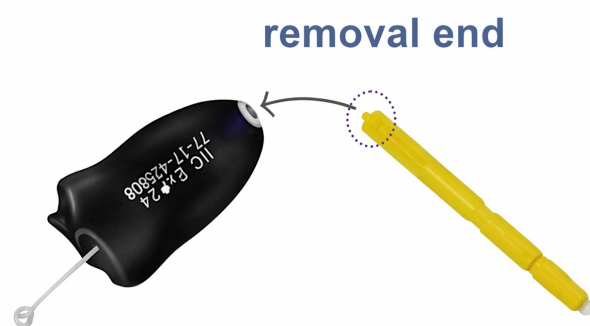

3

Use installed end to insert new filter.




4

Discard the used stick.



Helpful Tips

- Replace the filter when you notice reduced sound quality.
- Change filters regularly for the best sound and performance.
- Always use a new stick for each filter change.

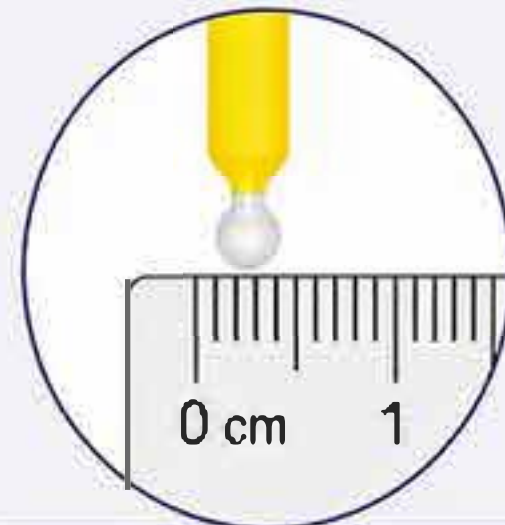

Each filter stick is

1.2 mm

Use gently—
do not force.



Absolute Hearing Solutions



750 Cross Pointe Road Suite F
Columbus, Ohio 43230



614-452-4280



www.absolutehearingsolutions.com



absolutehearing@att.net



Better Hearing.
Better Life.

Thank you for trusting us with your hearing health.