


ConnectClip

Quick Start Guide

Call
Absolute Hearing Solutions
For Hearing Aid Details!
614-452-4280
www.absolutehearingsolutions.com

PAIRING CONNECTCLIP WITH HEARING AIDS

1. Ensure ConnectClip is powered off. Press and hold multi-button for approximately 6 seconds until power indicator OFF ●
2. Turn both hearing aids OFF. (Open battery drawer)
3. Turn both hearing aids ON. (Close battery drawer)
4. The hearing aids will now be ready for first time connection with the ConnectClip. You now have 3 minutes to complete the pairing.
5. Turn ON ConnectClip. Press and hold multi-button for approximately 3 seconds until power indicator shows STEADY GREEN ●
6. ConnectClip will now search for your hearing aids. Status indicator BLINKING YELLOW ●
7. Place ConnectClip and both hearing aids close to each other (maximum distance is 8 inches)
8. Connection established with hearing aid(s) is indicated by status indicator STEADY YELLOW ●

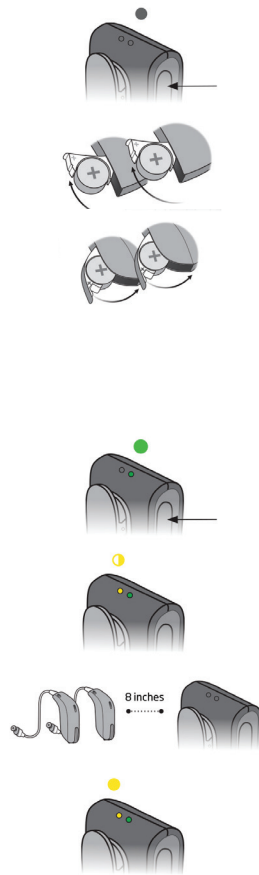

PAIRING CONNECTCLIP WITH MOBILE PHONE

1. Pair the ConnectClip with your hearing aids before pairing ConnectClip with your mobile phone.
2. Place ConnectClip in Pairing mode. Press and hold the multi-button AND volume up button for about 6 seconds (A) until status BLINKS BLUE ●
- The ConnectClip will be in pairing mode for about 3 minutes.
3. Enable pairing on mobile phone. Turn on Bluetooth. Search for and connect to/pair with ConnectClip. This is normally in the Settings/Bluetooth section of your phone.
4. Enter your pin code. If your phone requests a pin code enter 0000 (four zeros).
5. Pairing complete. Pairing complete with mobile phone is indicated by status STEADY BLUE ●



Absolute Hearing Solutions

Absolute Hearing Solutions LLC
absolutehearing@att.net
750 Cross Pointe Road Suite F
Gahanna, Ohio 43230
Phone: 614-452-4280
Toll Free: 888-803-2199
Fax: 614-577-0481
www.ab.solut.hearingsolutions.com

RECEIVING CALLS

Phone connected: When the ConnectClip is turned on and is in range of your mobile phone it will automatically connect to the mobile phone. The status indicator is STEADY BLUE ●


Phone ringing: When the phone rings you will hear a ring indicator (and, if available, the ring-tone from your phone) in your hearing aids. The status indicator will BLINK GREEN ●



Answer call: Either a short press of the multi-button (A) – or answer the call from your mobile phone.

Once the call is connected status will change to STEADY GREEN ●

End call: Either a short press of the multi-button (A) – or hang up the call from your mobile phone. Status STEADY BLUE ●

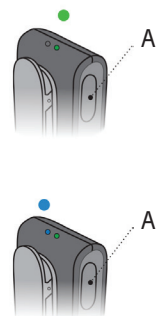

LISTENING TO MUSIC

If connected to a mobile phone or other paired device through Bluetooth, the music played on the phone/device will automatically be played in your hearing aids. STATUS ORANGE ●


You do not need to press any buttons or change anything on the ConnectClip.



REMOTE MICROPHONE

Press and hold multi-button for approximately 3 seconds to change mode. Mode change is confirmed by an audible indication in your hearing aids. STATUS MAGENTA ●


See "ConnectClip Quick Guide to the Remote Microphone" for more details.

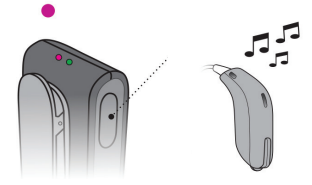

IMPORTANT: Ensure that the microphone is pointing to the speaker's mouth and that it is not covered by clothing, scarf, tie, etc.

REMOTE CONTROL

When ConnectClip is not streaming sound it doubles as a remote control for your hearing aids.

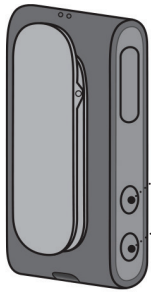

- **Volume up**
– short press
- **Volume down**
– short press



- **Next program**
– Press and hold for 3 seconds for next program
 - **Mute hearing aid microphones**
– long press on volume down button
- Short press volume up or down will **unmute**

IMPORTANT:

When used as a remote control, the maximum distance between ConnectClip and hearing aids is 6 feet.

Overview – ConnectClip blinking patterns

Status	Description
	Blinking yellow Searching for hearing aids
	Steady yellow Hearing aids connected. Not connected to a phone.
	Blinking blue Searching for mobile phone (pairing)
	Steady blue Connected to a mobile phone
	Steady green On a call
	Blinking green Incoming call
	Steady orange Music, Mobile Phone
	Steady magenta Remote microphone mode
	Blinking magenta Remote microphone reconnecting

Power	Description
	Steady red Battery low
	Steady green Turned on – not charging
	Blinking green Turned on – charging
	Fading green Turned on – charging complete
	Blinking orange Turned off – charging
	Steady orange Turned off – charging complete