


T² Remote Pocket User Guide

You may use a cell phone, cordless telephone, or corded telephone to adjust your hearing system. The telephone MUST "beep" when the keys are pressed for this function to work.

- Hold the phone approximately 3 inches from the ear
- Press ** to "wake" T² (a tone series indicates that it is ready for a command)
- T² will "sleep" after 20 seconds
- Press the telephone key that corresponds to the desired function

Use the following keypad buttons to remotely adjust your hearing aid settings.

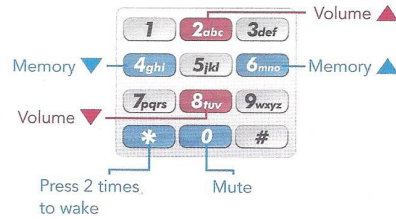

T² Remote Pocket User Guide

Troubleshooting Guide

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
	Telephone keys not emitting beep tones	Consult telephone operator's manual to enable beep tones
	T ² not unlocked prior to command	Press ** and listen for unique T ² indicator
Hearing system not responding to telephone command	Telephone too far from hearing system	Bring telephone closer to hearing system microphone
	Incorrect key command	Press ** to wake then press 2 or 8 to increase or decrease volume or 4 or 6 to change programs if device has multiple programs
Hearing system responded but won't respond to later command	T ² automatically "sleeps" 20 seconds after last command	Press ** to wake then press key that corresponds to desired action